

## **Competency Checklist for SMARTeZ® Elastomeric Pump**

Facility:	Date:	
Employee Name:	Title:	
Employee Department:		
Evaluator Name:	Title:	
Skill		Performed
Preparation		
Performs hand hygiene according to facility policy.		
2. Selects alcohol pads, IV flush syringe per facility protocol, and appropriat prescribed.	e SMARTeZ pump as	
3. Checks label on elastomeric pump for correct medication, dosage, and page	atient.	
Connects to access site		
4. Swabs and flushes vascular access site as directed by facility protocol.		
5. Connects the patient connector at the end of the tubing to the patient ac	cess site with the ON-	
OFF clamp closed. Ensures a secure connection.		
6. Opens the ON-OFF clamp to begin infusion and ensures the pump tubing	is free of kinks and the	
pump is secured safely near the patient.		
7. Documents the Infusion start time as directed by facility.		
Disconnect		
8. After the prescribed infusion duration, closes the ON-OFF clamp on the e	lastomeric pump and	
disconnects from the vascular access site.		
9. Flushes vascular access site as directed by facility policy.		
Comments:		
Signature of Employee:		